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
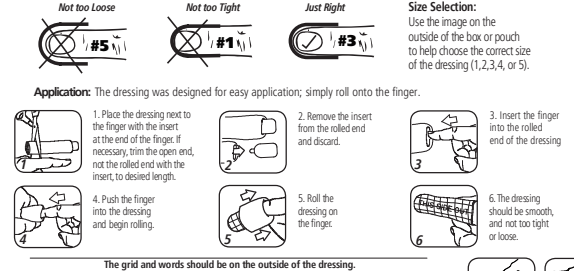
EN

PolyMem® Silver Finger/Toe Dressing

The dressing provides a soothing wrap shaped to fit.

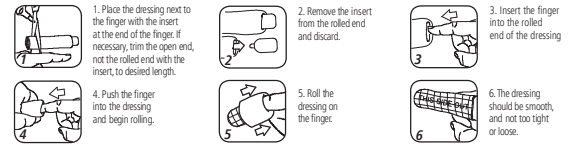Application : The dressing was designed for easy application; simply roll onto the finger.

The grid and words should be on the outside of the dressing.

The dressing should be snug, but maintain comfort. If it feels too tight, remove and apply a larger size. Remove by rolling off, opposite of application. The dressing may be applied in different ways. For toes the dressing may be cut along the sides creating flaps that can be laid upon the top and bottom of the foot. These can be secured with tape, or the dressing may be cut to form a ring or sleeve over the injured portion of the finger.

DESCRIPTION :

PolyMem polymeric membrane dressings are made of moisture-loving polyurethane matrix that contains (1) safe wound cleanser, (2) glycerol moisturizer, and (3) superabsorbents. Dressings have a thin polyurethane backing which breathes and keeps out contamination. Dressings may be used as primary, secondary, or combined primary and secondary dressing.

PolyMem Silver products incorporate very fine silver particles, and will not stain skin. Silver acts inside the dressing to protect from microbial contamination. Approximate silver content by area (µg/cm²) PolyMem Silver 124.

Drug-free PolyMem dressings help to:

• effectively manage and heal wounds
• absorb fluid and provide a moist healing environment
• minimize and relieve persistent and procedural wound pain
• reduce edema, bruising and the spread of inflammation into surrounding undamaged tissues

INDICATIONS :

Under the direction of a healthcare professional, for the management of serious wounds, such as most partial and full thickness wounds, including:

• Pressure ulcers (stages I-IV) • Acute wounds • Skin tears • 1st and 2nd degree burns • Leg ulcers • Diabetic ulcers • Surgical wounds
• Venous stasis ulcers • Donor and graft sites • Dermatologic disorders • Vascular ulcers

PRECAUTIONS :

• For external use only • PolyMem dressings are not compatible with oxidizing agents (hydrogen peroxide or hypochlorite solutions)
• Prior to performing imaging procedures, determine if the dressing should be removed • Avoid contact with electrodes or conductive gels
• Topical treatments are not recommended in conjunction with PolyMem dressings. • Be alert for signs of infection or maceration
• Do not use and discontinue use on people who show signs of sensitivity, irritation, or allergy from the dressing or its materials • Intended for one use on one person. Do not reuse. Reuse may degrade properties, spread contamination, and increase risk of infection

EN

PolyMem dressings may be used when signs of infection are present if proper medical treatment that addresses the cause of the infection has been implemented.

PolyMem Silver dressings are recommended when a wound displays delayed healing, signs of infection, or increased bioburden; or the patient has infection risk factors (such as contaminated wound, poor nutrition, diabetes, history of infection, etc.)

Store at room temperature ≤30°C (86°F).

Excursions permitted ≤55°C (131°F) but should be minimized.

Dispose as typical for biohazardous waste.

Dimensions are approximate.

DIRECTIONS :

INITIAL APPLICATION:

1. Prepare the injured area according to protocol or as directed by a physician or other ordering clinician. Clean the skin of dirt, topical medications and lotions. Retain moisture on the skin is beneficial.
2. Apply the dressing as outlined. A PolyMem dressing should also cover any inflamed or damaged area surrounding the injury. Dressings may be cut to accommodate the injury or body part.
3. Apply firm side out (so printing is visible).

Helpful Hints:

• Secure dressing with water-proof roll tape to reduce the chance of the inside of the dressing becoming contaminated.
• Outline the approximate wound margin on the outside of the dressing to help determine when to change.

Use:

1. A dramatic increase in wound fluid may be observed during the first few days. This is not uncommon, and indicates that the dressing is working.
2. Keep the dressing dry and in place when bathing or performing hand-hygiene. Change the dressing if it gets wet.

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© 2017, 2019 Ferris Mfg. Corp. or TM indicate trademark