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Diagrams showing application steps for fingers and toes. Includes icons for '너무 느슨하지 않게' (#5), '너무 조이지 않게' (#1), and '막 들어맞게' (#3). Includes numbered steps 1-6 for application and a section for '사용법' (Application) with '초 사용' (First use) and '재사용' (Reuse) instructions.

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Chinese text instructions for PolyMem bandage application. Includes diagrams for '不太松', '不太紧', '正合适'. Lists application steps 1-6 and includes '警告' (Warnings) and '注意' (Notes) sections.

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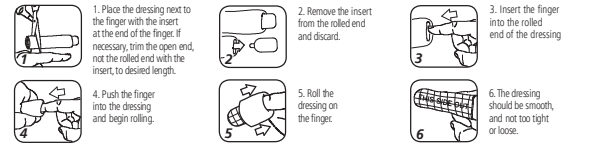
PolyMem® Finger/Toe Dressing

The dressing provides a soothing wrap shaped to fit.

Application: The dressing was designed for easy application; simply roll onto the finger.

The grid and words should be on the outside of the dressing.

The dressing should be snug, but maintain comfort. Application: The dressing may be applied in different ways...

DESCRIPTION :

PolyMem polymeric membrane dressings are made of moisture-loving polyurethane matrix that contains (1) safe wound cleanser, (2) glycerol moisturizer, and (3) superabsorbents.

Drug-free PolyMem dressings help to : • effectively manage and heal wounds • absorb fluid and provide a moist healing environment

INDICATIONS :

Indications: Over-the-counter, for management of minor wounds, such as : • Abrasions • Lacerations • Cuts • Burns Under the direction of a healthcare professional, for the management of serious wounds...

PRECAUTIONS :

• For external use only • PolyMem dressings are not compatible with oxidizing agents (hydrogen peroxide or hypochlorite solutions).

Do not use and discontinue use on people who show signs of sensitivity, irritation, or allergy from the dressing or its materials.

PolyMem dressings may be used when signs of infection are present if proper medical treatment that addresses the cause of the infection has been implemented.

Store at room temperature ≤30°C (86°F). Excursions permitted ≤55°C (131°F) but should be minimized.

DIRECTIONS :

Initial Application: 1. Prepare the injured area according to protocol or as directed by a physician or other ordering clinician.

Helpful Hints :

Secure dressing with water-proof roll tape to reduce the chance of the inside of the dressing becoming contaminated.

Use :

1. A dramatic increase in wound fluid may be observed during the first few days. This is not uncommon, and indicates that the dressing is working.

Dressing Change :

1. For an exuding wound, fluid will become visible through the top. Change before fluid reaches the wound margin, or when good practice dictates, or after no more than 7 days.

Observations during use or change :

During the normal healing process, the wound may appear to become larger during the first few dressing changes because the body is naturally removing non-viable tissue.

Observations during use or change :

During the normal healing process, the wound may appear to become larger during the first few dressing changes because the body is naturally removing non-viable tissue.

Observations during use or change :

During the normal healing process, the wound may appear to become larger during the first few dressing changes because the body is naturally removing non-viable tissue.